




Measurement Key

Length =

Circumference =

- Chest measurements are taken with patient lying down
- Pant measurements are taken with patient standing
- If in-between sizes refer to length measurements

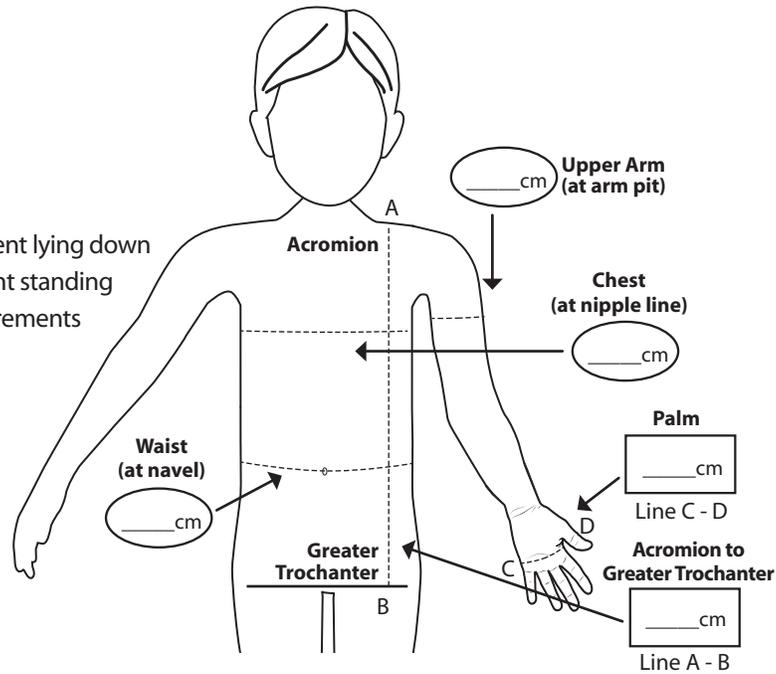

SPIO® Lower Body Orthosis

Waist circumference is measured at belly button.
Length is from waist to Ankle (P) or bottom of Patella (SP).

Short #/Long #

SP43 / P43	Waist Circumference: 43 cm, SP 27.30 cm long / P 40.33 cm long
SP45 / P45	Waist Circumference: 45 cm, SP 30.48 cm long / P 45.00 cm long
SP47 / P47	Waist Circumference: 47 cm, SP 32.00 cm long / P 47.95 cm long
SP49 / P49	Waist Circumference: 49 cm, SP 33.02 cm / P 49.80 cm long
SP51 / P51	Waist Circumference: 51 cm, SP 34.29 cm / P 52.70 cm long
SP53 / P53	Waist Circumference: 53 cm, SP 36.83 cm / P 56.84 cm long
SP55 / P55	Waist Circumference: 55 cm, SP 39.37 cm / P 63.19 cm long
SP57 / P57	Waist Circumference: 57 cm, SP 42.54 cm / P 66.60 cm long
SP59 / P59	Waist Circumference: 59 cm, SP 45.30 cm long / P 70.04 cm long
SP61 / P61	Waist Circumference: 61 cm, SP 46.00 cm long / P 71.70 cm long
SP63 / P63	Waist Circumference: 63 cm, SP 48.89 cm long / P 77.80 cm long
SP65 / P65	Waist Circumference: 65 cm, SP 50.16 cm long / P 79.30 cm long
SP67 / P67	Waist Circumference: 67 cm, SP 54.70 cm long / P 81.70 cm long
SP69 / P69	Waist Circumference: 69 cm, SP 55.25 cm long / P 83.20 cm long
SP71 / P71	Waist Circumference: 71 cm, SP 57.50 cm long / P 86.20 cm long
SP73 / P73	Waist Circumference: 73 cm, SP 59.50 cm long / P 90.50 cm long

SPIO® Classic TLSO

Chest circumference is measured at nipple line.
Length is from Acromion to Greater Trochanter.

Item #

V41	Chest Circumference: 41 cm, 27 cm long
V44	Chest Circumference: 44 cm, 30.75 cm long
V47	Chest Circumference: 47 cm, 34.5 cm long
V51	Chest Circumference: 51 cm, 38.25 cm long
V54	Chest Circumference: 54 cm, 44 cm long
V57	Chest Circumference: 57 cm, 47.75 cm long
V60	Chest Circumference: 60 cm, 51.5 cm long
V64	Chest Circumference: 64 cm, 55.25 cm long
V67	Chest Circumference: 67 cm, 59 cm long
V70	Chest Circumference: 70 cm, 62.75 cm long
V73	Chest Circumference: 73 cm, 66.5 cm long
V77	Chest Circumference: 77 cm, 70.25 cm long

SPIO® Arm Orthosis

Arm circumference is measured at arm pit.

Item #

AS13	Arm Circumference: 13 cm, 19.05 cm long
AS14	Arm Circumference: 14 cm, 20.06 cm long
AS16	Arm Circumference: 16 cm, 22.86 cm long
AS17	Arm Circumference: 17 cm, 24.07 cm long
AS18	Arm Circumference: 18 cm, 28.00 cm long
AS20	Arm Circumference: 20 cm, 30.48 cm long
AS21	Arm Circumference: 21 cm, 32.04 cm long
AS22	Arm Circumference: 22 cm, 35.56 cm long
AS24	Arm Circumference: 24 cm, 38.10 cm long
AS25	Arm Circumference: 25 cm, 40.00 cm long
AS26	Arm Circumference: 26 cm, 42.50 cm long
AS28	Arm Circumference: 28 cm, 45.72 cm long

SPIO® Upper Body Orthosis

Chest circumference is measured at nipple line.

Length is from Acromion to Greater Trochanter.

Short #/Long #

SS41/ S41	Chest Circumference: 41 cm, 23.00 cm long
SS44/ S44	Chest Circumference: 44 cm, 27.94 cm long
SS47/ S47	Chest Circumference: 47 cm, 31.11 cm long
SS51/ S51	Chest Circumference: 51 cm, 35.56 cm long
SS54/ S54	Chest Circumference: 54 cm, 39.00 cm long
SS57 / S57	Chest Circumference: 57 cm, 45.72 cm long
SS60/ S60	Chest Circumference: 60 cm, 47.00 cm long
SS64/ S64	Chest Circumference: 64 cm, 50.50 cm long
SS67/ S67	Chest Circumference: 67 cm, 56.40 cm long
SS70/ S70	Chest Circumference: 70 cm, 60.50 cm long
SS73/ S73	Chest Circumference: 73 cm, 64.30 cm long
SS77/ S77	Chest Circumference: 77cm, 68.50 cm long

SPIO® Wrist-Hand Orthosis

Palm width is measured at widest part above thumb.

Item #

G5	Width of Palm: 5 cm, 11.43 cm long
G5.5	Width of Palm: 5.5 cm, 12.19 cm long
G6	Width of Palm: 6 cm, 13.03 cm long
G6.5	Width of Palm: 6.5 cm, 14.04 cm long
G7	Width of Palm: 7 cm, 15.55 cm long
G7.5	Width of Palm: 7.5 cm, 16.50 cm long
G8	Width of Palm: 8 cm, 17.02 cm long

FabriFlex Wrap

Wraps can be used for many applications. These are the measurements of the Wraps only. Smaller wraps are good for younger children and upper limbs. Larger wraps are good for older children and lower limbs.

Item #

F720	7 inch x 20 inch
F930	9 inch x 30 inch
F1135	11 inch x 35 inch

FabriFlex Wrap

Straps can be used for many applications. These are the dimensions of the straps only. Smaller straps are good for younger children and upper limbs. Larger straps are good for older children and lower limbs.

Item #

FP02	Two inch 3 pack (2 inch x 30, 40, and 50 inch)
FP04	Four inch 2 pack (4 inch x 30 and 50 inch)